High-Q-Bond Band™

LIGHT CURE BAND CEMENT

EN

PROPERTIES

Bonds metal bands to tooth surfaces. Uses light cure adhesive technology to provide you with additional working time to ensure accurate band placement.

Small particle glass filled material designed specifically for orthodontic use. Releases Fluoride.

Contains the adhesive promoter 4-META, for strong reliable chemical bonding with tooth, metal and porcelain surfaces.

INSTRUCTIONS FOR USE

1. Clean teeth with plain pumice using a rotating bristle brush or rubber cup.
2. Rinse and air-dry.
3. Isolate the bonded surfaces.
4. Apply a small amount of **High-Q-Bond Band Cement** onto the tooth-bearing surfaces of the band.
5. Immediately seat the band onto the tooth with a slight rotating motion. A small amount of cement should exude around the entire band periphery. Final adjustment must be made before the cement begins to cure from the ambient light (approximately 60 seconds).
6. Light cure entire bonded tooth surface for 20 seconds from two opposite directions.
7. **High-Q-Bond Band Cement** can be easily wiped away with a cotton roll before it starts setting, or can be removed with a scaler after it has set.

PRECAUTIONS!

Refrigeration will increase viscosity, making syringing more difficult, and will retard setting time. Contains polymerizable monomers, which may cause allergic contact dermatitis in susceptible patients. Wash thoroughly with soap and water after contact. If skin sensitization occurs, or if known allergy to methacrylate resin exists, discontinue use.

ESSENTIAL INGREDIENTS

- Urethan dimethacrylate.
- Triethylen glycoldimethacrylate.
- Multifunctional monomers.
- Photoinitiators.
- Fumed Silica.
- Adhesive accelerator.

STORAGE

- Do not store above 24°C (75°F).
- Keep away from direct sunlight.
- For optimum freshness, keep refrigerated.
- Do not freeze.
- To obtain SDS visit www.bjmlabs.com.
- Do not use after expiration date. See expiry date on the label.
- Shelf life is 2 years after the date of manufacturing.

 **B.J.M. Laboratories Ltd.**
12 Hassadna St., Industrial Park,
Or-Yehuda 6022011, Israel
tel. 972-74-7000111 fax. 972-3-7353020
Info@bjmlabs.com www.bjmlabs.com

QUASER S.r.l.
Via Pian due Torri 19, 00146 Rome, Italy
tel. 39-06-55282325 fax. 39-06-55290821
info@quasersrl.it www.quasersrl.it

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