Q-Ceram™

CERAMIC PRIMER

EN

INDICATIONS

B.J.M. Q-Ceram is a surface conditioner for ceramic materials such as porcelain and zirconia restorations.

PROPERTIES

B.J.M. Q-Ceram is a one bottle primer material, for surface preparation of ceramic crowns, prior to bonding with resin cements. This primer improves cementation of zirconia crowns and does not require additional conditioning such as sandblasting. Its low viscosity and excellent wetting properties promote ideal surface coverage.

IT IS USED FOR

- Preparation of ceramic crowns, veneers and inlays prior to cementation.
- Preparation of fractured ceramic crowns for repair with resin materials.

PREPARING CERAMIC SURFACE FOR BONDING

1. Clean ceramic surface with alcohol and dry well with oil-free air.
2. Open Q-Ceram bottle and dispense 1-2 drops into mixing well, replace bottle cap immediately.
3. Use applicator to apply generous amount of primer onto ceramic surfaces.
4. Wait 1 minute.
5. Dry gently for 20 seconds with oil-free air.
6. Continue with cementation.

CAUTION!

Avoid contact with skin, oral soft tissues and eyes. If accidental contact occurs, rinse with copious amount of water and seek medical attention immediately.

ESSENTIAL INGREDIENTS

- Ethanol.
- Water.
- Multifunctional monomers.
- Adhesive accelerator.

STORAGE

- Do not store above 24°C (75°F).
- For optimum freshness, keep refrigerated.
- Do not freeze.
- Tightly recap after use.
- To obtain SDS visit www.bjmlabs.com.
- Do not use after expiration date. See expiry date on the label.
- Shelf life is 2 years after the date of manufacturing.

B.J.M. Laboratories Ltd.
12 Hassadna St., Industrial Park,
Or-Yehuda 6022011, Israel
tel. 972-74-7000111 fax. 972-3-7353020
Info@bjmlabs.com www.bjmlabs.com

QUASER S.r.l.
Via Pian due Torri 19, 00146 Rome, Italy
tel. 39-06-55282325 fax. 39-06-55290821
info@quasersrl.it www.quasersrl.it

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