# Q-Etch™

# Q-Etch UF™

## EN ETCHING GEL - PHOSPHORIC ACID 37%

Q-Etch contains phosphoric acid for etching tooth enamel and dentin when preparing surfaces for the application of composites.

### PROPERTIES

- High quality, non-drying, medium viscosity gel.
- Stays put, does not run.
- Large, comfortable-to-use syringe.
- Tip can be angled to facilitate easy & accurate placement of the gel even in distal & lingual composite preparations. Washes off quickly & easily.
- Has excellent water solubility.
- Visible color.
- Disposable tips save time and are more sanitary.

### INSTRUCTIONS FOR USE

BJM Q-Etch can be applied precisely in controlled amounts to the tooth enamel and dentin area to be etched. The effective recommended etching time is: Enamel - 20 to 30 seconds. Dentin - As indicated by the manufacturer of the bonding agent. The Gel Etch should then be rinsed off with copious quantities of water for at least 20 seconds.

### DRYING

Enamel - Dry adequately until the surface has a frosty white appearance. Dentin - As indicated in the instruction for use of the bonding agent.

### CAUTION!

1. Q-Etch contains phosphoric acid. Avoid contact with skin and eyes. If accidental contact occurs, flush with copious amounts of water.
2. The disposable tip should be removed and discarded after each use! As a Re-use can cause cross-contamination. ☹
3. Re-Cap syringe after each use, to prevent drying of the Etch Gel.

### ESSENTIAL INGREDIENTS

- Q-Etch:**
- Phosphoric acid 37%.
  - Fumed Silica.
  - Water.
  - Pigment.

### Q-Etch UF:

- Phosphoric acid 37%.
- Thickening agent.
- Water.
- Pigment.

### STORAGE

- Do not store above 24°C (75°F).
- For optimum freshness, keep refrigerated.
- Do not freeze.
- Tightly recap after use.
- To obtain SDS visit [www.bjmlabs.com](http://www.bjmlabs.com).
- Do not use after expiration date. See expiry date on the label.
- Shelf life is 2 years after the date of manufacturing.

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