High-Q-Bond SE™

EN SELF-ETCH / SELF-ADHESIVE DUAL CURED PERMANENT RESIN CEMENT

High-Q-Bond SE is permanent resin cement does not require etching, priming or bonding. Self / light cured. The "Automix" system provides a rapid, consistent and homogeneous mixture which can be dispensed directly into the resation or tooth cavity.

INSTRUCTIONS FOR USE

1. Remove the provisional restoration. Note: Use only temporary cements that are eugenol-free. The use of eugenol-containing temporary materials can inhibit the polymerization process of the High-Q-Bond SE.
2. Rinse thoroughly.
3. Air dry the preparation (do not desiccate).
4. Try in the restoration to ensure proper fit.
5. Bleed the syringe before initial use. Place mixing tip on the Automix syringe.
6. Dispensing the material can be done as follows: Inlay/Onlay • Mount the supplied intra oral tip onto the mixing tip and dispense the cement directly

covering all surfaces. Seat the restoration gently onto the preparation allowing the cement to flow from all sides. Crown • Dispense the cement directly into the restoration covering a urfaces. Seat the restoration gently onto the preparation allowing the cement to flow from all margins. Post • Mount the supplied XXF intra oral tip onto the mixing tip. Syringe the cement into the canal. Seat the post and vibrate slightly avoiding possibility of trapped air. Allow the cement to slowly flow from the canal space.

7. When the restoration is properly seated, remove all excess cement. Excess cement is best removed in its gel state with a scaler or explorer. Gel state can be achieved by tack curing excess with a light for 2 seconds or allowing to self-cure for 1.5 - 2 minutes after application. For best results, leave undisturbed for 1.5 minutes before tack curing.
8. TIMES: • Working time is 2 minutes. • Oral set

time is 3 minutes. • Light-curing time: 20 seconds (see manufacturer's recommendation). • After removal of excess cement, light cure all surfaces including margins for 20 seconds.

9. Use finishing strips for proximal areas. Polish margins with polishing discs, cups, or points.
10. Check occlusion. Note: Automix tips are designed for single patient use only.

CAUTION!

Uncured methacrylate resin may cause contact dermatitis and damage the pulp. Avoid contact with skin, eyes and soft tissue. Wash thoroughly with water after contact.

ESSENTIAL INGREDIENTS

Base & Catlyst:

- 4-Methacryloxyethyltrimellitic anhydride .
- Multifunctional monomers.
- Urethane dimethacrylate.

- Photoinitiators.
- Polymerization activator.
- Glass filler.
- Fumed Silica.

STORAGE

- Do not store above 24°C (75°F).
- Keep away from direct sunlight.
- For optimum freshness, keep refrigerated.
- Do not freeze.
- If refrigerated, allow the syringe to reach room temperature.
- To obtain SDS visit www.bjmlabs.com.
- Do not use after expiration date. See expiry date on the label.
- Shelf life is 2 years after the date of manufacturing.

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