Q-Ceram™
Ceramic primer

For professional dental use only

EN INDICATIONS

- Q-Ceram is a surface conditioner for ceramic materials such as porcelain and zirconia restorations.
- Used for preparation of ceramic crowns, veneers, and inlays prior to cementation.
- Used for preparation of fractured ceramic crowns for repair with resin materials.

PROPERTIES

- Q-Ceram is a single-step Primer material, for surface preparation of ceramic crowns, prior to bonding with resin cements.
- Q-Ceram improves cementation of zirconia crowns and does not require additional conditioning such as sandblasting.
- Its low viscosity and excellent wetting properties promote ideal surface coverage.
- Q-Ceram has been on the market since 2014 with no changes in formula. No complaints of any health-related or chemical performance issues were received.

CAUTION!

- Avoid contact with skin, oral soft tissues and eyes. If accidental contact occurs, rinse with ample amount of water and seek medical attention immediately.
- Q-Ceram contains materials which: highly flammable / irritating to respiratory system and skin. For complete symbols guide see below.

COMPOSITION

Essential ingredients	% wt.
Ethanol	<90%
10-methacryloyldodecylphosphonate	<5%
3-trimethoxysilylpropyl methacrylate	<5%

FILLER CONTENT

The product does not contain filler.

INSTRUCTIONS FOR USE

1. Clean ceramic surface and leave it moist.
2. Dispense 1-2 drops of Q-Ceram into a dappen dish, re-cap bottle immediately after use.
3. Use applicator to apply generous amount of Q-Ceram onto ceramic surfaces. Leave undisturbed for 1 minute.
4. Dry gently for 20 seconds with oil-free air.
5. Continue with cementation.

STORAGE AND DISPOSAL

- Do not store above 24°C (75°F).
- Keep product in original packaging.
- Keep away from direct sunlight.
- For optimum freshness, keep refrigerated.
- Do not freeze.
- To obtain SDS or IFU visit www.bjmlabs.com.
- Do not use after expiration date. See expiry date on the label.
- Shelf life is 2 years after the date of manufacturing.
- Empty product can be safely disposed after use in an intended regular waste container. The product does not require special disposal instructions or precautions and is harmless to the environment.

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