Q-Etch™

Q-Etch UF™

Q-Etch™ / Q-Etch UF™

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EN INDICATIONS

- Q-Etch / Q-Etch UF contains phosphoric acid for etching tooth enamel and dentin for surfaces preparation.
- Q-Etch can be used for any procedure that requires etching.

PROPRIÉTÉS

- Q-Etch is a 37% Phosphoric Acid Etching Gel with excellent viscosity for precise application.
- Available in 2 consistencies: Regular viscosity (Q-Etch) and Unfilled Silica Free (Q-Etch UF) viscosity.
- High quality, non-drying, medium viscosity etching gel.
- Stays put, does not run.
- Large, comfortable-to-use syringe.
- Tip can be angled to facilitate easy and accurate placement of the gel even in distal and lingual composite preparations.
- Washes off quickly and easily.
- Has excellent water solubility.
- Blue visible color. To see shade guide visit www.bjmlabs.com.
- Disposable tips save time and are more sanitary.
- Q-Etch / Q-Etch UF has been on the market since 2000 with no changes in formula. No complaints of any health-related or chemical performance issues were received.

CAUTION!

- Q-Etch / Q-Etch UF contains phosphoric acid. Avoid contact with skin and eyes. If accidental contact occurs, flush with copious amounts of water.
- Q-Etch / Q-Etch UF contains materials which can cause burns if it incorrectly used. For complete symbols guide see below.

COMPOSITION

Essential ingredients Q-ETCH	% WT.	Essential ingredients Q-ETCH UF	% WT.
Phosphoric acid	37%	Phosphoric Acid	37%
Water	<60%	Water	<60%
Fumed silica	<15%	Thickening agent	<10%
Pigment	<0.1%	Pigment	<0.1%

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