**Zirconite™**  
Dual-cure permanent adhesive resin cement designed for zirconia

For professional dental use only

**EN INDICATIONS**

- Permanent cementation of Ceramic and Zirconia restorations.
- Proven Color Stability over time.
- PROPRIÉTÉS**
  - A novel dual-cure permanent adhesive resin cement specially formulated for Zirconia cementation.
  - ISO 4049 classification: Type 2, Class 3.
  - Self-etch, self-adhering, dual cured, radiopaque permanent resin cement and does not require etching, priming, or bonding.
  - Radiopacity value above 2.5 mm Aluminium. (Radiopacity is defined as the ability of a material to be visible in x-ray photographs.)
  - Chemical affinity for metal and Zirconia.
  - Dual-cure cement – it has a self curing mechanism which can also be light cured for an immediate set and extra strength at the margins.
  - Features high retention and total margin integrity.
  - Administration via Automic syringes saves application time and guarantees consistent mix.
  - Available in 3 shades: Dentin, White, Translucent. To see shade guide visit [www.bjmlabs.com](http://www.bjmlabs.com).
  - Zirconite has been on the market since 2009 with no changes in formula. No complaints of any health-related or chemical performance issues were received.

**CAUTION!**

- Contains polymerizable monomers which may cause allergic contact dermatitis in susceptible patients and damage the pulp. Avoid contact with skin, eyes, and soft tissue. Wash thoroughly with water after contact.
- If skin sensitization occurs or if known allergy to methacrylate resin exists, discontinue use.
- Contains materials which: sensitive to light / irritating to eyes / irritating to respiratory system and skin / may cause sensitization by skin contact. For complete symbols guide see below.

**COMPOSITION**

Essential ingredients - BASE	% wt.	Essential ingredients - CATALYST	% wt.
Composition of acrylic resins	<20%	Composition of Acrylic Resins	<30%
2-Hydroxyethylmethacrylate	<10%	Triethylenglycoldimethacrylate	<20%
Triethylenglycoldimethacrylate	<10%	Sodium p-toluenesulfinate	<1%
Photoinitiators	<5%	Fumed Silica	<5%
Copper (II) chloride dihydrate (3% aqueous solution)	<1%	Glass filler	<60%
10-methacryloyloxydecyl dihydrogen phosphonate	<10%		
Fumed silica	<5%		
Glass filler	<60%		

**FILLER CONTENT**

The filler system comprises glass fillers and modified silica:

50-70 weight-% or 35-50 volume-%.

The particle size of the fillers range: &lt;6 um.

**INSTRUCTIONS FOR USE**

- Note:**
- Tips are for single patient use only to prevent cross-contamination between patients.
  - Use one Detachable Manual Adapter per one syringe lifetime.
1. Remove the provisional restoration.
- Note: Use only temporary cements that are eugenol free. The use of eugenol-containing temporary luting materials can inhibit the polymerization process of Zirconite.

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