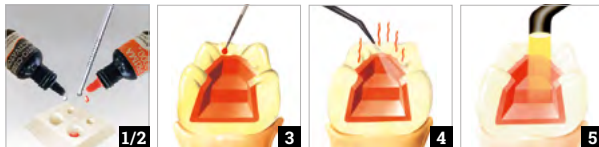
Auto-Cure Activator™

Auto-Cure Activator™

Chemical activator for use with Prima 2000 and Prima Quick Bond

For professional dental use only

EN INDICATIONS

Auto-Cure Activator is used for self-curing of Prima 2000 and Prima Quick Bond in:

- Adhesive cementation of crown and bridge, inlays and onlays.
- Bonding amalgam restorations.
- Light-inaccessible procedures.

PROPERTIES

- Auto-Cure Activator is a chemical activator for use with Prima 2000 and Prima Quick Bond.
- Auto-Cure Activator has been on the market since 2003 with no changes in formula. No complaints of any health-related or chemical performance issues were received.

CAUTION!

- Auto-Cure Activator contains acetone and polymerizable monomers which may cause allergic contact dermatitis in susceptible persons. Avoid contact with eyes and skin. Wash thoroughly with water after contact. If skin sensitization occurs, or if known allergy to methacrylate resin exists, discontinue use.
- Auto-Cure Activator contains materials which: highly flammable / irritating to eyes / may cause sensitization by skin contact / repeated exposure may cause skin dryness or cracking / vapors may cause drowsiness and dizziness. For complete symbols guide see below.

COMPOSITION

Essential ingredients	% wt.	Essential ingredients	% wt.
Acetone	<80%	Benzoyl Peroxide	<5%
Ethanol	<20%		

FILLER CONTENT

The product does not contain filler.

INSTRUCTIONS FOR USE

1. Dispense equal number of drops of Prima 2000 / Multi Bond / Prima Quick Bond and Auto-Cure Activator in a clean dappen dish.
2. Mix for 2 seconds with a clean, unused brush or microbrush.
3. Using the same brush, apply ample amount of the mixture on tooth surface and leave undisturbed for 15 seconds.
4. Remove solvents with a gentle air flow for at least 15 seconds or until no visible movement of the resin is noted. If the surface appears partially coated, apply additional material and dry again.
5. Finish restoration or cementation.
6. Tightly re-cap bottle after use.

STORAGE AND DISPOSAL

- Do not store above 24°C (75°F).
- Keep product in original packaging.
- Keep away from direct sunlight.
- For optimum freshness, keep refrigerated.
- Do not freeze.
- To obtain SDS or IFU visit www.bjmlabs.com.
- Do not use after expiration date. See expiry date on the label.
- Shelf life is 2 years after the date of manufacturing.
- Empty product can be safely disposed after use in an intended regular waste container. The product does not require special disposal instructions or precautions and is harmless to the environment.

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