High-Q-Bond Retainer™

High-Q-Bond Retainer™ Light-cure retainer

For professional dental use only

EN INDICATIONS

- Direct bonding of orthodontic lingual retainers.
- Direct bonding of orthodontic clear aligners attachments.

PROPERTIES

- High-Q-Bond Retainer bonds chemically and mechanically for a superior bond.
 - High-Q-Bond Retainer is a light-cure material - compatible with all curing units available on the market.
 - High-Q-Bond Retainer depth of cure is >4 mm.
 - The viscosity of High-Q-Bond Retainer was designed to prevent drifting.
 - Orange visible color. To see shade guide visit www.bjmlabs.com.
 - High-Q-Bond Retainer has been on the market since 2007 with no changes in formula.
- No complaints of any health-related or chemical performance issues were received.

CAUTIONS!

- Refrigeration will increase viscosity, making syringing more difficult and will retard setting time.
- High-Q-Bond Retainer contains polymerizable monomers, which may cause allergic contact dermatitis in susceptible patients. Wash thoroughly with soap and water after contact. If skin sensitization occurs, or if known allergy to methacrylate resin exists, discontinue use.
- Polymerization may be triggered by ambient lighting or the dental operating light. Tightly recap the syringe after use.
- High-Q-Bond Retainer contains materials which: sensitive to light / irritating to eyes / harmful if swallowed / irritating to respiratory system and skin / may cause sensitization by skin contact. For complete symbols guide see below.

COMPOSITION – Essential ingredients

HIGH-Q-BOND RETAINER	% wt.	Q-ETCH ORTHO	% wt.
Urethan dimethacrylate	<20%	Phosphoric acid	37%
Triethyleneglycoldimethacrylate	<20%	Water	<60%
2-hydroxyethylmethacrylate	<5%	Thickening agent	<10%
Polymerization activators	<1%	Pigment	<0,1%
Photoinitiators	<1%		
Glass filler	<60%		
Fumed silica	<5%		

FILLER CONTENT

High-Q-Bond Retainer:

The filler system comprises glass fillers and modified silica:

60-70 weight-% or 42-50 volume-%.
The particle size of the fillers ranges: <6 µm.

Q-Etch Ortho:

The product does not contain filler.

INSTRUCTIONS FOR USE

Bonding of orthodontic lingual retainers:

1. Clean teeth with plain pumice using a rotating bristle brush or rubber cup.
2. Rinse and air-dry.
3. Etch for 20 seconds using Q-Etch Ortho or any other 36-38% phosphoric acid etching gel. Rinse and air-dry. High-Q-Bond Primer or any other orthodontic primer can be used after etching.

4. Isolate the bonded surfaces.
5. Position retainer on teeth and syringe the High-Q-Bond Retainer directly onto the splint and adjacent teeth surfaces.
6. Final retainer adjustment must be made before High-Q-Bond Retainer begins to cure from the ambient light (approximately 60 seconds).
7. High-Q-Bond Retainer can be easily wiped away with a cotton roll before it starts setting or removed with a scaler after it has set. The strong orange color of the uncured cement helps to identify excess material.
8. Light-cure each bonded tooth surface for 20 seconds. (Cure only for 10 seconds with high intensity lights.)
9. After light-curing, the photo chromatic High-Q-Bond Retainer will change its color from orange to white. This serves as an indicator for complete polymerization of the retainer.
10. Tightly re-cap syringe after use.

Bonding of orthodontic clear aligners attachments:

1. Clean teeth with plain pumice using a rotating bristle brush or rubber cup. Rinse and air-dry.
2. Use the clear aligner template to see the precise placement of the attachments and etch the attachment area for 20 seconds with Q-Etch Ortho (or any other 36-38% phosphoric acid etching gel available on the market). Rinse and air-dry.
3. Apply High-Q-Bond Bracket Primer (or any other orthodontic primer available on the market) on the etched surface only. Light-cure for 20 sec.
4. Apply High-Q-Bond Retainer into the clear aligner, use the syringe to compress the bonding material to avoid air-bubbles. Remove excess material. High-Q-Bond Retainer can easily be wiped away with a cotton roll before it starts setting or removed with a scaler after it has set. The strong orange color of the uncured composite helps to identify excess material.
5. Check the clear aligner with the bonding material.
6. Place the clear aligner over the teeth. Hold the clear aligner firmly in place, and light-cure for 2x20 sec. After light-curing, the photo chromatic High-Q-Bond Retainer will change its color from orange to white. This serves as an indicator for complete polymerization of the retainer.
7. Tightly re-cap syringe after use.
8. Gently remove attachment template. Remove any residual High-Q-Bond Bracket Primer or Retainer flash with a scaler.

	min:sec
Light curing in a light-wave range of 440 – 470 nm	00:20
Compatible with all curing units available on the market	

STORAGE AND DISPOSAL

- Do not store above 24°C (75°F).
- Keep product in original packaging.
- Keep away from direct sunlight.
- For optimum freshness, keep refrigerated.
- Do not freeze.
- If refrigerated, allow the syringe to reach room temperature.
- To obtain SDS or IFU visit www.bjmlabs.com.
- Do not use after expiration date. See expiry date on the label.
- Shelf life is 2 years after the date of manufacturing.
- Empty product can be safely disposed after use in an intended regular waste container. The product does not require special disposal instructions or precautions and is harmless to the environment.

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