Q-Crown™

Temporary self-cure crown & bridge material

For professional dental use only

EN INDICATIONS

- Q-Crown is used to fabricate temporary crowns, bridges, inlays and onlays used in indirect restorative procedures.
- Temporary crown and bridge fabrication can be made either directly or indirectly with impression trays.
- Proven Color Stability over time.

PROPERTIES

- Q-Crown is a restorative material.
- Q-Crown classification according to ISO 4049 is Type 1, Class 1.
- Q-Crown is a controlled radical self-curing, paste-paste composite, for fabrication of temporary crowns and bridges.
- Easy to remove during its elastic phase.
- Does not heat up during polymerization.
- Great color stability
- Easy to finish and polish.
- High biocompatibility.
- Suitable for long-term temporary crowns and bridges due to its high mechanical strength and precision fitting.
- Q-Crown is available in 3 shades: A1, A2, A3. To see shade guide visit www.bjmlabs.com.
- Q-Crown has been on the market since 2013 with no changes in formula. No complaints of any health-related or chemical performance issues were received.

ATTENTION

- Do not use Q-Crown if patient has a history of severe allergic reaction to acrylate or methacrylate.
- Any compromise in oral hygiene or use of some mouth washes may lead to discoloration.

PRECAUTIONS

- Before use, lubricate the contacting hard surfaces to prevent breakage when the temporary is removed. Composite or Glass Ionomer cores must always be lubricated (e.g., with glycerin or Vaseline) to prevent them from adhering to the Q-Crown material.

CAUTION!

- Q-Crown contains polymerizable monomers. Avoid prolonged or repeated contact with skin, oral soft tissues and eyes. Avoid prolonged inhalation. Do not take internally. If skin sensitization occurs, or if known allergy to methacrylate resin exists, discontinue use.
- Q-Crown contains materials which: irritating to eyes / irritating to respiratory system and skin / may cause sensitization by skin contact. For complete symbols guide see below.

COMPOSITION

Essential ingredients Q-Crown BASE	% wt.	Essential ingredients Q-Crown CATALYST	% wt.	Essential ingredients Q-Crown GLAZE	% wt.
Triethyleneglycoldimethacrylate	<30%	Triethyleneglycoldimethacrylate	<20%	Bisphenol A-glycidyl Methacrylate	<30%
Bisphenol a-glycidyl methacrylate	<30%	Bisphenol A-glycidyl methacrylate	<35%	Urethane Dimethacrylate	<40%
Polymerization activators	<10%	Polymerization activators	<10%	Triethyleneglycoldimethacrylate	<60%
Glass filler	<40%	Glass filler	<40%	Polymerization activators	<1%
Fumed silica	<10%	Fumed silica	<5%		

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FILLER CONTENT

Q-CROWN

The filler system comprises glass fillers and modified silica: 40-50 weight-% or 10-15 volume-%. The particle size of the fillers ranges: <6 um.

Q-CROWN GLAZE

The product does not contain filler.

INSTRUCTIONS FOR USE

Note:

- Before use, lubricate the contacting hard surfaces to prevent breakage when the temporary is removed. Composite or Glass Ionomer cores must always be lubricated (e.g., with glycerin or Vaseline) to prevent them from adhering to the Q-Crown material.
- Tips are for single patient use only to prevent cross-contamination between patients.

- Bleed the syringe before installation of the automix syringe mix tip. Mount a new, automix syringe mix tip onto the dual-barrel syringe and bleed the syringe again.
- Q-Crown reaches its elastic phase 60-90 seconds after start of mix, i.e., about 30-60 seconds after insertion in mouth. The impression must be removed from the mouth during this phase otherwise it may become difficult to remove. For the material time indications, see table below.
- During working time, the material should be removed from the mouth.
- Moderate setting occurs at between 2:00 – 2:30 minutes from start of mix. After moderate setting remove the material from the impression tray, remove the thin inhibited layer with alcohol (this makes the finishing easier) and carefully remove any excess or rough material in areas close to the gums.
- Finishing and polishing are done only after the material has completely set.
- Cement the temporary crown using a non-eugenol temporary cement (e.g. Q-Temp).
- Brush a thin layer of the included Q-Crown Glaze on to the temporary crown and light-cure each surface for 20 seconds.
- Tightly re-cap syringe after use.
- Time indications:
The working and setting times depend on the ambient and oral temperature.
The times shown are based on conditions relevant for practice.

	min:sec
Working time at ambient temperature	>01:30
Intra-oral setting time	<05:00

STORAGE AND DISPOSAL

- Do not store above 24°C (75°F).
- Keep product in original packaging.
- Keep away from direct sunlight.
- For optimum freshness, keep refrigerated.
- Do not freeze.
- If refrigerated, allow the syringe to reach room temperature.
- To obtain SDS or IFU visit www.bjmlabs.com.
- Do not use after expiration date. See expiry date on the label.
- Shelf life is 2 years after the date of manufacturing.
- Empty product can be safely disposed after use in an intended regular waste container. The product does not require special disposal instructions or precautions and is harmless to the environment.

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B.J.M. Laboratories Ltd.