Q-Etch Ortho™

Etching gel - Phosphoric acid 37%

For professional dental use only

EN INDICATIONS

- Q-Etch Ortho contains phosphoric acid for etching tooth enamel and dentin for surfaces preparation.
- Q-Etch Ortho can be used for any procedure that requires etching.

PROPERTIES

- Q-Etch Ortho is a 37% Phosphoric Acid Etching Gel with excellent viscosity for precise application.
- High quality, non-drying, medium viscosity etching gel.
- Stays put, does not run.
- Large, comfortable-to-use syringe.
- Tip can be angled to facilitate easy and accurate placement of the gel even in distal and lingual composite preparations.
- Washes off quickly and easily.
- Has excellent water solubility.
- Turquoise visible color. To see shade guide visit www.bjmlabs.com.
- Disposable tips save time and are more sanitary.
- Q-Etch Ortho has been on the market since 2000 with no changes in formula. No complaints of any health-related or chemical performance issues were received.

CAUTION!

- Q-Etch Ortho contains phosphoric acid. Avoid contact with skin and eyes. If accidental contact occurs, flush with copious amounts of water.
- Q-Etch Ortho contains materials which can cause burns if it incorrectly used. For complete symbols guide see below.

COMPOSITION

Essential ingredients	% wt.
Phosphoric acid	37%
Water	<60%

Essential ingredients	% wt.
Thickening agent	<10%
Pigment	<0.1%

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FILLER CONTENT

The product does not contain filler.

INSTRUCTIONS FOR USE

Note:

Delivery tips are for single patient use only to prevent cross-contamination between patients.

Application of Q-Etch Ortho:

Q-Etch Ortho can be applied precisely in controlled amounts to the tooth enamel and dentin area to be etched.

The effective recommended etching time is:

Enamel – 20 to 30 seconds.

Dentin – As indicated by the manufacturer of the bonding agent.

The Gel Etch should then be rinsed off with copious quantities of water for at least 20 seconds.

Tightly re-cap syringe after use.

Drying:

Enamel – Dry adequately until the surface has a frosty white appearance.

Dentin – As indicated in the instruction for use of the bonding agent.

STORAGE AND DISPOSAL

- Do not store above 24°C (75°F).
- Keep product in original packaging.
- Keep away from direct sunlight.
- For optimum freshness, keep refrigerated.
- Do not freeze.
- If refrigerated, allow the syringe to reach room temperature.
- To obtain SDS or IFU visit www.bjmlabs.com.
- Do not use after expiration date. See expiry date on the label.
- Shelf life is 2 years after the date of manufacturing.
- Empty product can be safely disposed after use in an intended regular waste container. The product does not require special disposal instructions or precautions and is harmless to the environment.

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