Q-Glass Ortho™
Glass-ionomer band cement

For professional dental use only

EN INDICATIONS

- Bonding orthodontic bands.

PROPERTIES

- Strong chemical adhesion.
- Fluoride release.
- Low sensitivity to moisture.
- Blue color for good visibility.
- Q-Glass has been on the market since 2004 with no changes in formula. No complaints of any health-related or chemical performance issues were received.

CAUTION!

- Q-Glass Ortho liquid and Q-Glass Ortho mixed cement paste are corrosive. Avoid contact with skin, oral soft tissues and eyes. In case of contact with oral tissue or skin, remove immediately with alcohol-soaked cotton wool and rinse thoroughly with water. In case of contact with eyes, immediately rinse thoroughly with water and seek medical attention.
- Do not mix Q-Glass Ortho powder or liquid with any other glass ionomer-based component system. Direct contact with eugenol-based products is to be avoided. Eugenol inhibits the setting of glass ionomer sealers.
- Q-Glass Ortho contains materials which: irritating to eyes / irritating to respiratory system and skin. In the event of an allergic reaction, consult a specialist. For complete symbols guide see below.

COMPOSITION

Essential ingredients POWDER	% wt.	Essential ingredients LIQUID	% wt.
Strontium aluminosilicate glass powder	<90%	Polyacrylic acid	<40%
Polyacrylic acid	<10%	Tartaric acid	<20%
		Water	<70%

FILLER CONTENT IN MIXED PASTE

The filler system comprises glass fillers and modified silica: 80-90 weight-% or 30-40 volume-%. The particle size of the fillers range: <6 um.

INSTRUCTIONS FOR USE

1. Prepare the tooth according to the usual protocol.

2. Clean the tooth surface with pumice and water, proph cup or brush.
3. Before dispensing, fluff the powder bottle by shaking it up and down for 5 seconds and shake the liquid bottle. For accurate dispensing of liquid drops, hold the bottle vertically upside down. Allow air bubbles to escape before dripping. If the nozzle is clogged, clear with damp gauze. Discard drops with air bubbles.
4. Mix the necessary quantity of cement. The recommended dosage is 1 spoon of powder for 4 drops of liquid, which corresponds to 2.0 gr powder and 1.1 gr liquid. For a liquid consistency, use slightly less powder.
Powder/Liquid ratio (g/g) 2.0 / 1.1
5. Apply one level scoop of powder and 4 drops of liquid on a mixing pad.
6. Divide the powder into 2 equal portions. Mix one portion with the liquid for about 10 seconds. Add the second portion of powder and mix for 10-15 seconds. Total mixing time should not exceed 20-25 seconds.
7. Apply cement on interior of band in the usual manner. Seat band on tooth.
Note: High temperatures shot working time.
8. Working time is 3 minutes from the onset of mixing. Remove excess cement. Wires can be attached after a setting time of 5 minutes.
9. Tightly re-cap the bottles after use.

Time indications:

The working and setting times depend on the ambient and oral temperature. The times shown are based on conditions relevant for practice.

	min:sec
Mixing time	00:25
Working time	2:30-3:00
Setting time	3:00-3:30

STORAGE AND DISPOSAL

- Do not store above 24°C (75°F).
- Keep product in original packaging.
- Keep away from direct sunlight.
- For optimum freshness, keep refrigerated.
- Do not freeze.
- If refrigerated, allow the syringe to reach room temperature.
- To obtain SDS or IFU visit www.bjmlabs.com.
- Do not use after expiration date. See expiry date on the label.
- Shelf life is 2 years after the date of manufacturing.
- Empty product can be safely disposed after use in an intended regular waste container. The product does not require special disposal instructions or precautions and is harmless to the environment.

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