Q-Seal™

Light-cure, pit & fissure sealant

For professional dental use only

EN INDICATIONS

- Pediatric sealant applications.
- For the preventive sealing of pits and fissures.

PROPERTIES

- Q-Seal is a light-cure material - compatible with all curing units available on the market.
- Q-Seal classification according to ISO 4049 is Type 2, Class 2.
- Easy and precise application by using direct syringe technique.
- Low viscosity, excellent flow properties, optimal wetting of fissures.
- Q-Seal depth of cure is >5 mm.
- Q-Seal has been on the market since 2006 with no changes in formula. No complaints of any health-related or chemical performance issues were received.

CONTRAINDICATION

Sealing of pits and fissures with Q-Seal is contradicted:

- If pits or fissures are already affected with caries.
- If a dry working field cannot be established during the application and curing of Q-Seal.

ATTENTION

Do not use Q-Seal if the patient is known to be allergic to any of the ingredients. For complete ingredients guide see below.

CAUTION!


- Q-Seal contains polymerizable monomers, which may cause allergic contact dermatitis in susceptible persons. Wash thoroughly with soap and water after contact. If skin sensitization occurs, or if known allergy to methacrylate resin exists, discontinue use.
- Polymerization may be triggered by ambient lighting or the dental operatory light. Tightly recaps the bottle after use.
- Q-Seal contains materials which: sensitive to light / irritating to eyes / harmful if swallowed / irritating to respiratory system and skin / may cause sensitization by skin contact. For complete symbols guide see below.

COMPOSITION

Essential ingredients – Q-SEAL	% wt.	Essential ingredients – Q-ETCH	% wt.
Urethane dimethacrylate	<60%	Phosphoric acid	37%
Triethylenglycoldimethacrylate	<40%	Water	<60%
4-methacryloxyethyltrimellitlic anhydride	<5%	Fumed silica	<15%
Photoinitiators	<1%	Pigment	<0,1%
Fumed silica	<3%		
Multifunctional monomers	<5%		

FILLER CONTENT

The filler system comprises of modified silica:
1-3 weight-% or 0.1-0.6 volume-%.
The particle size of the fillers range: <6 um.

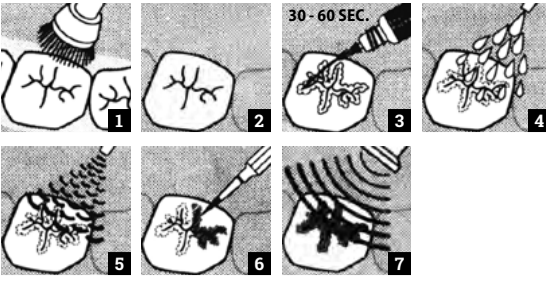Q-Etch

The filler system comprises modified silica:
10-15 weight-% or 2-3 volume-%.
The particle size of the fillers ranges from 0.2 to 0.3 um.

INSTRUCTIONS FOR USE

- Nota:**
- Tips are for single patient use only to prevent cross-contamination between patients.**
1. Clean the enamel surface thoroughly with either an oil and fluoride free cleansing paste, or with pumice and brush.
 2. Isolate the working field (the use of rubber dam is recommended).
 3. Apply Q-Etch etching gel and let it react for 30 to 60 seconds.
 4. Rinse thoroughly.
 5. Dry with water and oil free air. The etched enamel should have a mat white appearance. Avoid contamination of the etched surface with saliva.
 6. Moisten and secure lock a new dispensing tip on the Q-Seal syringe and carefully syringe an adequate amount of Q-Seal into the fissures. Let the sealant flow until the entire fissure is covered. You can use a disposable brush, microbrush or the dispensing tip itself to spread the sealant and eliminate air bubbles.
 7. Light-cure sealant for 20 sec. Check occlusion.
 8. Tightly re-cap syringe after use.

	min:sec
Light curing in a light-wave range of 390 – 470 nm	00:20
Compatible with all curing units available on the market	

STORAGE AND DISPOSAL

- Do not store above 24°C (75°F).
- Keep product in original packaging.
- Keep away from direct sunlight.
- For optimum freshness, keep refrigerated.
- Do not freeze.
- If refrigerated, allow the syringe to reach room temperature.
- To obtain SDS or IFU visit www.bjmlabs.com.
- Do not use after expiration date. See expiry date on the label.
- Shelf life is 2 years after the date of manufacturing.
- Empty product can be safely disposed after use in an intended regular waste container. The product does not require special disposal instructions or precautions and is harmless to the environment.

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