## Q-Etch™ / Q-Etch UF™

### Etching gel - Phosphoric acid 37%

*For professional dental use only.*

*Patient Population: Any human undergoing dental treatment related to the procedures listed below under "Indications". The definitive patient group is diagnosed on a case-by-case basis by a licensed dental professional and there are no exclusions for specific patient groups.*

#### EN INDICATIONS

- Q-Etch / Q-Etch UF contains phosphoric acid for etching tooth enamel and dentin for surfaces preparation.
- Q-Etch can be used for any procedure that requires etching.

#### PROPERTIES

- Q-Etch is a 37% Phosphoric Acid Etching Gel with excellent viscosity for precise application.
- Available in 2 consistencies: Regular viscosity (Q-Etch) and Unfilled Silica Free (Q-Etch UF) viscosity.
- High quality, non-drying, medium viscosity etching gel.
- Stays put, does not run.
- Large, comfortable-to-use syringe.
- Tip can be angled to facilitate easy and accurate placement of the gel even in distal and lingual composite preparations.
- Washes off quickly and easily.
- Has excellent water solubility.
- Blue visible color. To see shade guide visit [www.bjmlabs.com](http://www.bjmlabs.com).
- Disposable tips save time and are more sanitary.
- Q-Etch / Q-Etch UF has been on the market since 2000 with no changes in formula. No complaints of any health-related or chemical performance issues were received.

#### CONTRAINDICATIONS

- Not to be used on exposed pulp or any other soft tissue.
- The use of the medical device is contraindicated if the patient is known to be allergic or sensitive to phosphoric acid.
- The use of the medical device is contraindicated if the stipulated application technique cannot be applied.

#### CAUTION!

- Q-Etch / Q-Etch UF contains phosphoric acid. Avoid contact with skin and eyes. If accidental contact occurs, flush with copious amounts of water.
- Q-Etch / Q-Etch UF contains materials which can cause burns if it incorrectly used. For complete symbols guide see below.
- Any serious incident that occurs in relation to this device should be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient is established. You may also contact our EU Authorised Representative: Obelis S.A. (contact details below).

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#### COMPOSITION

Essential Ingredients Q-ETCH	.WT %	Essential Ingredients Q-ETCH UF	.WT %
Phosphoric acid	37%	Phosphoric Acid	37%
Water	<60%	Water	<60%
Fumed silica	<15%	Thickening agent	<10%
Pigment	<0.1%	Pigment	<0.1%

#### FILLER CONTENT

**Q-Etch**  
The filler system comprises modified silica:  
10-15 weight-% or 2-3 volume-%.  
The particle size of the fillers ranges from 0.2 to 0.3 um.

#### Q-Etch UF

The product does not contain filler.

#### INSTRUCTIONS FOR USE

##### Note:

Delivery tips are for single patient use only to prevent cross-contamination between patients.

##### Application of Q-Etch / Q-Etch UF:

Q-Etch / Q-Etch UF can be applied precisely in controlled amounts to the tooth enamel and dentin area to be etched.  
The effective recommended etching time is:  
Enamel – 20 to 30 seconds.  
Dentin – As indicated by the manufacturer of the bonding agent.  
The Gel Etch should then be rinsed off with copious quantities of water for at least 20 seconds.  
Tightly re-cap syringe after use.

##### Drying:

Enamel – Dry adequately until the surface has a frosty white appearance.  
Dentin – As indicated in the instruction for use of the bonding agent.

##### STORAGE AND DISPOSAL

- Do not store above 24°C (75°F).
- Keep product in original packaging.
- Keep away from direct sunlight.
- For optimum freshness, keep refrigerated.
- Do not freeze.
- If refrigerated, allow the syringe to reach room temperature.
- To obtain SDS or IFU visit [www.bjmlabs.com](http://www.bjmlabs.com).
- Do not use after expiration date. See expiry date on the label.
- Shelf life is 2 years after the date of manufacturing.
- Empty product can be safely disposed after use in an intended regular waste container. The product does not require special disposal instructions or precautions and is harmless to the environment.

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