Q-Etch™ / Q-Etch UF™ Etching gel - Phosphoric acid 37%

For professional dental use only.

Patient Population: Any human undergoing dental treatment related to the procedures listed below under "Indications". The definitive patient group is diagnosed on a case-by-case basis by a licensed dental professional and there are no exclusions for specific patient groups.

INDICATIONS

- Q-Etch / Q-Etch UF contains phosphoric acid for etching tooth enamel and dentin for surfaces preparation.
- Q-Etch can be used for any procedure that requires etching.

PROPERTIES

- Q-Etch is a 37% Phosphoric Acid Etching gel with excellent viscosity for precise application.
- Available in 2 consistencies: Regular viscosity (Q-Etch) and Unfilled Silica Free (Q-Etch UF) viscosity.
- High quality, non-drying, medium viscosity etching gel.
- Stays put, does not run.
- Large, comfortable to-use syringe.
- Tip can be angled to facilitate easy and accurate placement of the gel even in distal and lingual composite preparations.
- Washes off quickly and easily.
- Has excellent water solubility.
- Blue visible color. To see shade guide visit www.bjmlabs.com.
- Disposable tips save time and are more sanitary.
- Q-Etch / Q-Etch UF has been on the market since 2000 with no changes in formula. No complaints of any health-related or chemical performance issues were received.

CONTRAINDICATIONS

- Not to be used on exposed pulp or any other soft tissue.
- The use of the medical device is contraindicated if the patient is known to be allergic or sensitive to phosphoric acid.
- The use of the medical device is contraindicated if the stipulated application technique cannot be applied.

CAUTIONS

- Q-Etch / Q-Etch UF contains phosphoric acid. Avoid contact with skin and eyes. If accidental contact occurs, flush with copious amounts of water.
- Q-Etch / Q-Etch UF contains materials which can cause burns if incorrectly used. For complete symbols guide see below.
- Any serious incident that occurs in relation to this device should be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient is established. You may also contact our EU Authorised Representative: Obelis S.A. (contact details below).

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COMPOSITION

Essential ingredients Q-ETCH	WT.%	Essential ingredients Q-ETCH UF	WT.%
Phosphoric acid	37%	Acide phosphorique	37%
Water	<60%	Water	<60%
Fumed silica	<15%	Thickening agent	<10%
Pigment	<0.1%	Pigment	<0.1%

FILLER CONTENT

The product does not contain filler.

The particle size of the fillers ranges from 0.2 to 0.3 µm.

Q-Etch UF

The product does not contain filler.

The particle size of the fillers ranges from 0.2 to 0.3 µm.

INSTRUCTIONS FOR USE

The product does not contain filler.

The particle size of the fillers ranges from 0.2 to 0.3 µm.

Notes:

Delivery tips are for single patient use only to prevent cross-contamination between patients.

Application of Q-Etch / Q-Etch UF: Q-Etch UF can be applied precisely in controlled amounts to the tooth enamel and dentin area to be etched.

The effective recommended etching time is:

Enamel – 20 to 30 seconds.

The gel Etch should then be rinsed off with copious quantities of water for at least 20 seconds.

Drying:

Etch – Dry adequately until the surface has a frosty white appearance.

Dentin – As indicated in the instruction for use of the Bonding agent.

STORAGE AND DISPOSAL

- Do not store above 24°C (75°F).
- Keep product in original packaging.
- Keep away from direct sunlight.
- For optimum freshness, keep refrigerated.
- Do not freeze.
- If refrigerated, allow the syringe to reach room temperature.
- To obtain SDS or IFU visit www.bjmlabs.com.
- Do not use after expiration date. See expiry date on the label.
- Shelf life is 2 years after the date of manufacturing.
- Empty product can be safely disposed after use in an intended regular waste container. The product does not require special disposal instructions or precautions and is harmless to the environment.

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