Porcelain Fix™

Porcelain Fix™

Porcelain preparation kit

For professional dental use only.

Patient Population: Any human undergoing dental treatment related to the procedures listed below under "Indications". The definitive patient group is diagnosed on a case-by-case basis by a licensed dental professional and there are no exclusions for specific patient groups.

EN INDICATIONS

- Porcelain Fix is used for bonding resin to porcelain.
- Used for preparation of porcelain crowns, veneers and inlays prior to cementation.
- Used for preparation of fractured porcelain or porcelain fused to metal crowns for bonding.

PROPERTIES

- Porcelain Fix is a complete kit for preparing porcelain veneers, inlays and fractured porcelain teeth for bonding. It consists of Porcelain Etch and Porcelain Silane.
- Porcelain Etch is a specially buffered, viscous, hydrofluoric acid that will etch porcelains of all type to produce a microscopic porous surface, which will provide a strong mechanical interlock with composite resin materials.
- Porcelain Silane is a single component Silane that enhances bonding values and durability when applied to etched porcelain surfaces.
- Porcelain Fix has been on the market since 1998 with no changes in formula. No complaints of any health-related or chemical performance issues were received.

CONTRAINDICATIONS

- Not to be used on Tooth Surfaces.
- Avoid contact with Oral Mucosa.
- The use of the medical device is contraindicated if the patient is known to be allergic or sensitive to any of its ingredients.

CAUTION!

- Porcelain Etch must be used carefully and cautiously. Avoid contact with oral soft tissues, skin and eyes. If accidental contact occurs, flush with copious amounts of water and seek medical attention immediately. In case of Intraoral procedure, we strongly recommend to isolate the area selected to be etched with a rubber dam.
- Porcelain Fix contains materials which: highly flammable / irritating to eyes / irritating to respiratory system and skin and can cause burns if it incorrectly used. For complete symbols guide see below.
- Any serious incident that occurs in relation to this device should be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient is established. You may also contact our EU Authorised Representative: Obelis S.A. (contact details below).

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COMPOSITION

Essential ingredients – ETCH	% wt.	Essential ingredients – SILANE	% wt.
Water	<90%	Ethanol	<90%
Hydrofluoric acid	<10%	Stabilizer	<10%
Thickening agent	<5%		

FILLER CONTENT

The product does not contain filler.

INSTRUCTIONS FOR USE

- In case of Intraoral procedure, isolate the area to be etched with a rubber dam.
- When preparing fractured porcelain crowns for bonding, abrade the surface of the porcelain to be etched with a medium grit bur to remove the glazed surface and to make space for a sufficient layer of composite resin, for strength and aesthetics. Rinse all debris from the porcelain surface with copious amounts of water and dry with moisture and oil free compressed air.
- Syringe the Porcelain Etch onto the porcelain surface. Leave undisturbed for 60 seconds.
- Rinse the porcelain thoroughly with water to completely remove the etchant and dry well. The etched porcelain should have a frosted white appearance, if not, repeat the etching procedure.
- Brush Porcelain Silane on the etched porcelain surface. Dry well with a stream of moisture and oil free compressed air.
- Tightly re-cap bottles and or syringes after use.
- Apply composite resin of your choice and finish.

Note: Porcelain Fix bonding steps are also illustrated in High-Q-Bond illustration cards 3 and 4 enclosed separately with this package.

STORAGE AND DISPOSAL

- Do not store above 24°C (75°F).
- Keep product in original packaging.
- Keep away from direct sunlight.
- For optimum freshness, keep refrigerated.
- Do not freeze.
- To obtain SDS or IFU visit www.bjmlabs.com.
- Do not use after expiration date. See expiry date on the label.
- Shelf life is 2 years after the date of manufacturing.
- Empty product can be safely disposed after use in an intended regular waste container. The product does not require special disposal instructions or precautions and is harmless to the environment.

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