

Assembly Instructions for Manual Adapter for Automix Syringe

EN



1. Keep the rib of the Detachable Manual Adapter aligned with the notch of the Dual Barrels Syringe



2. Put the Detachable Manual Adapter onto Dual Barrels Syringe




3. Remove the Cap from the Detachable Manual Adapter



4. The Detachable Manual Adapter is ready for use



5. Put the Cap back onto the Detachable Manual Adapter

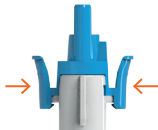

6. Press both flanks of Detachable Manual Adapter and then detach it from the Dual Barrel Syringe

Attention: Use one Adapter per one syringe life-time